

User Manual
of
Detachable Basket Stretcher
TC104




Operating the Basket Stretcher

Assemble the basket stretcher

1. To pack the basket stretcher, it should be divided into two parts to save space.
2. When use, it should be assembled.
3. The pipe on the bottom of one part on basket stretcher should be aligned with the relative hole on the other part of the basket stretcher.




4. After install the basket stretcher, there should no crevice between the two parts.

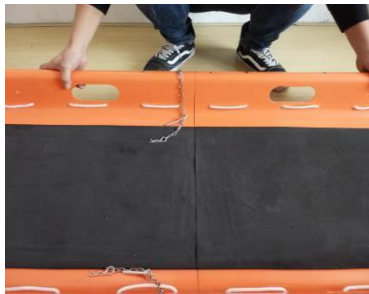


5. Also there be the fixed device to make sure the basket stretcher safety.

6. There is the screw. One screw be fixed on one part of the basket stretcher. Another screw is aside. When connect the two parts of the basket stretcher, the aside screw is useful.


7. The aside screw should be installed into the hole on the basket stretcher.

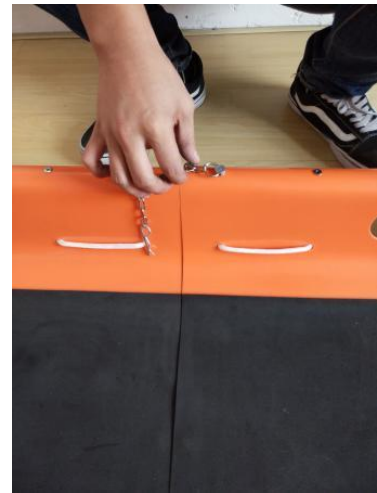


8. On the bottom of the screw, there is hole.
9. There is fixed device to help fix the screw to keep safe.

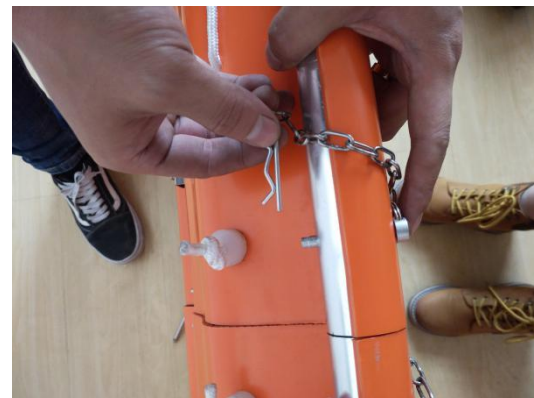

10. On the bottom of the basket stretcher, there is also fixed device.
11. To install the long pole to the hole.

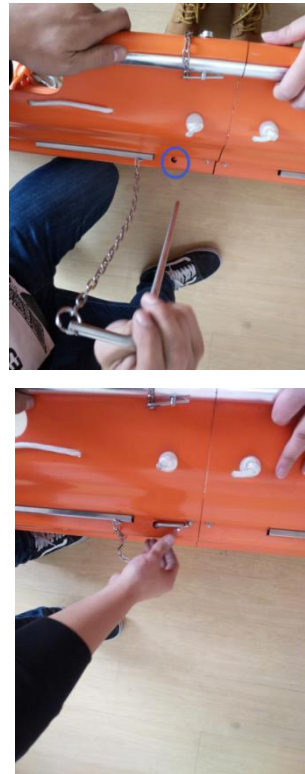

12. At the one side of the pole, there is the hole to fix the fixed device.
13. For the fixed device, it could install as the photo shows.

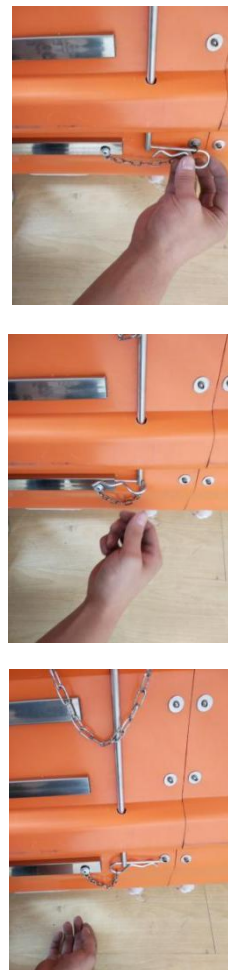

Install the safety belts device on the basket stretcher



13. As the basket stretcher could be used on helicopter, there is the safety belts devices.

14. There has the screw and shim to install in the small hole of the basket stretcher.


15. It could be installed as the right photo shows.



16. After install the shim fixed device, it should be the same as the right photo shows.

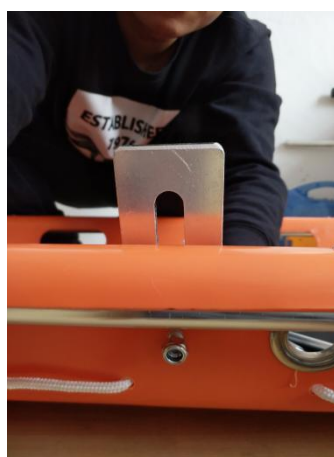
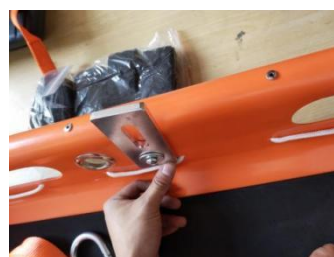


17. Also there is the woven belt to install in the hole.

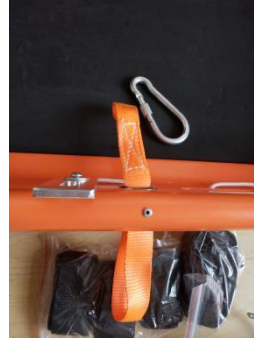

18. It should be hook up the woven belt and the shim fixed device by the hook.

19. Then the long belts which connected to the helicopter should be fixed to the hook.

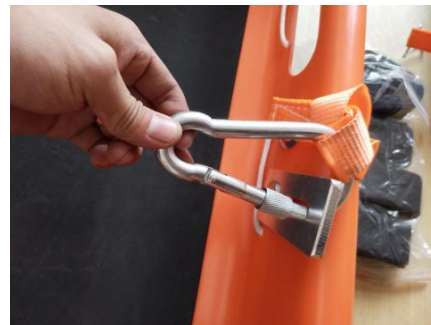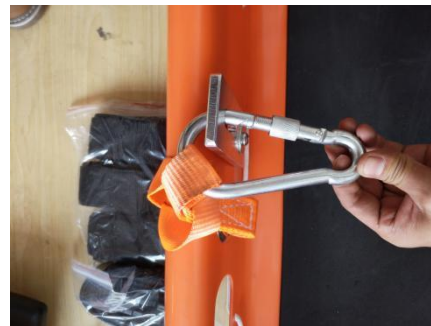

Install the Foot Pad of the basket stretcher

stretcher


20. There is a foot pad for different people.

21. To install the woven belt with hook into the hole. And make sure the woven belt just inlaid the groove.



22. There is the hook to connect with the basket stretcher.

23. There are three hooks to fit for different people.

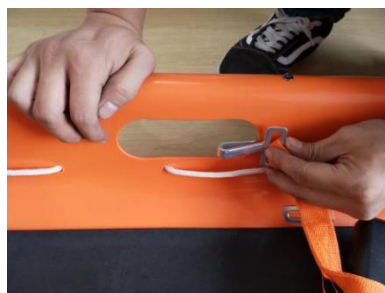

24. After install the foot pad, it will be like the photo show.
25. Also its location is adjustable.



Cleaning

Clean the basket stretcher and its parts with a clean cloth or soft brush and warm soapy water. Rinse with warm water and air dry. Do not clean with products comprising of bleach or phenol. A stiff bristled brush can be used if necessary.